Product description

FotoClean is a ready to use, water-based cleaning mixture for use in the Bandelin ultrasonic cleaner with built-in heating for cleaning of additively manufactured components.

Processing

Preparation

Shake well before each use! FotoClean must be homogeneously mixed before use.

Instructions for use

1. Pour FotoClean into the bath/into the inserted beakers of the Bandelin ultrasonic cleaner with built-in heating (REF. 82100) and make sure the cleaner is warmed up to the temperature specified in the instructions of the material.
2. Place the components to be cleaned free of contact with other objects to be cleaned in the insert basket of the ultrasonic cleaning device/ in the beaker.
3. Refer to the instructions of the material for further cleaning parameters.
4. Refer to the instructions of the material for the cleaning parameters for post-cleaning with water.

Note: Used cleaning fluid should be homogenized by stirring before the ultrasonic cleaner is switched on again. Regular replacement of the cleaning fluids in the event of reductions in cleaning performance is recommended for optimal cleaning results.

FotoClean is only suitable for use in the Bandelin RK100H ultrasonic cleaning device. No liability is accepted for any damage by the use of other devices.

Risks and safety information

Observe the safety data sheet when using FotoClean.

Storage

Storage at 18–28 °C

Batch number / Use-by date

The batch number and use-by date are marked every pack of FotoClean. When submitting a complaint about a product, please always communicate this information. Do not use the product after expiration of the use-by date.

Deviations from the process described may result in changes to the properties of FotoClean.

Disposal

The used cleaning fluids should be disposed of in Europe in accordance with the disposal class EN / substance(s): A110111. In other countries, please consult the local authorities and disposal companies.

FotoClean

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Directions for use

Dreve Dentamid GmbH is not liable for any damages caused by improper application of the cleaner.

Dreve Dentamid GmbH
 Max-Planck-Straße 31
 59423 Unna/Germany
www.dentamid.dreve.com

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Die Dreve Dentamid GmbH haftet nicht für Schäden, die durch fehlerhafte Anwendung des Reinigers hervorgerufen werden.