Uso previsto

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Product description

Both GumQuick materials have been conceived on the basis of addition-vulcanising vinyl polysiloxanes for the production of flexible gingiva masks onto the master model.

Indication for use

GumQuick: Gingiva mask made in the dental laboratory for C&B. GumQuick attains a final hardness of 60 Shore A and reproduces the natural proportions in combination with the opaque gingiva color. Therefore unphysiological pressure onto the gingiva parts can already be avoided during the wax modellation.

GumQuick implant: Gingiva mask made in the dental laboratory for highly precise implant works. GumQuick implant has been developed as special gingiva mask material for the implant technique. The high final hardness of 75 Shore A and the low flexibility of the material guarantee a safe and easy modellation of the secondary parts taking into consideration the natural course of the gingiva.

Processing

GumQuick (indirect method)

A matrix of hard kneadable silicone material (e.g. Laborsil pro) is to be applied onto the dies of the unsawn die model which should be provided with the gingiva mask. After curing the kneadable silicone should be cut so that the top of the dies will appear (approx. 1 mm). The model can be sawn and gingiva parts ground as usual. Afterwards the correct position of the matrix has to be checked. In order to guarantee a faultless surface of the gingiva mask Separator

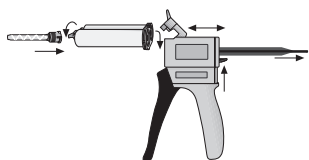
should be applied onto the contact surface kneadable silicone / GumQuick.

GumQuick is to be applied directly from the cartridge onto the model and into the matrix. Now the matrix has to be positioned at the model so that the surplus of material can flow out through the holes at the top of the die. Ideally, the gingiva mask should be cured under pressure. After a period of approx. 6:30 minutes the matrix can be eliminated and the mask can be treated with scissors, scalpel or with rotating grinding instruments.

GumQuick implant (direct method)

At first DuoSep is to be applied onto the contact surface impression material / GumQuick implant in order to guarantee a faultless surface of the gingiva mask.

Attention: In order to avoid a damage of the gingiva mask surface when applying GumQuick implant in conjunction with polyether impressions DuoSep is to be used! Before the production of the gingiva mask the polyether impressions should ventilate for at least 3 hours. Afterwards GumQuick implant is to be applied directly from the cartridge into the impression in the area of the impression posts. Thanks to the ideal flowing characteristics of GumQuick implant the abutments will be surrounded safely. After a period of approx. 6:30 minutes GumQuick implant is cured and the impression can be filled with plaster. The gingiva mask can easily be treated with scissors, scalpels or with rotating grinding instruments.

This product was developed for dental use and has to be used according to the instructions. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. Furthermore the user is bound to check the product according to suitability and possible

use in self dependence before using it subject to its purpose, especially in case the purpose is not mentioned in the instructions.

Caution

- The product is exclusively intended for the above-mentioned range of use and shall only be used by professionally qualified and instructed persons.
- Polymerisation, referring to the setting of cross-linked silicones, can be disrupted by latex gloves, creams, plastics, cleaning agents etc. For this reason, we recommend that you wear commercially available polyethylene gloves.
- The setting times can be increased by lower temperatures or reduced by higher temperatures.
- Wear protective clothing! Set material is chemically resistant. Marks on your clothing cannot be removed.
- Pre-convey the material before attaching the mixing cannula.
- Never convey to a mixing cannula which has been cured.
- After use, leave the mixing cannula on the double cartridge as a closure.

Keep away from children!

Batch number/use-by date

The batch number and the use-by date are located on both the outer packaging and the product. When submitting a complaint about a product, please always quote the batch number. Do not use the product after expiration of the use-by date.

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