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### Lieferform

<b>[REF]</b> <b>D4304</b>	1,0 Liter Flasche
<b>[REF]</b> <b>D4303N</b>	5,0 Liter Kanister

### Product description

Isolat Film is a heat-resistant, film-forming insulation agent on the basis of alginate for the field of application plaster against plaster as well as plaster against resin.

### Processing instructions

Apply Isolat Film onto the freshly set plaster model resp. onto a well moistened old model with a soft brush in one coating. Possible overdoses have to be removed carefully before the drying of Isolat Film. The drying time amounts to 5–10 minutes, depending on the degree of moisture. Prewarmed models accelerate the drying of Isolat Film considerably. When reusing older dry models it is recommended to boil out the model and to coat it newly with Isolat Film.

### Important advice

- The product is exclusively designed for the above mentioned field of application and is only to be used by qualified and instructed personnel.
- Dilution is possible with water (better Aqua dest.). Close well after use.

The product has been developed for the dental field and has to be processed according to the patient information. For damages which can result from ulterior use or faulty processing the manufacturer does not assume any liability. Furthermore, the user is bound to check the material to aptitude and

**Keep out of the reach of children! Only for dental use!**

### Storage conditions

Store in a dark place at room temperature, i. e. 18 °C–28 °C.

**Shelf life:** 2 years in the closed original container

### Delivery form

<b>[REF]</b> <b>D4304</b>	1.0 l bottle
<b>[REF]</b> <b>D4303N</b>	5.0 l canister

purpose before use, especially when the purpose is not mentioned in the patient information.

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