**Doelbepaling**

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**Intended use**

Lightplast base plates is intended for dental use:

- for the production of individual impression trays for precision impressions for accurately fitting restorations with the highest marginal accuracy, e. g.
  - functional impressions
  - situational impressions for unfavorable dental arch and jaw shapes and/or deep-set bands
- for production of a occlusal base plate
- for intraoral support pin registration

The user is obliged to examine the material for suitability and usage possibilities for the intended purposes on his own responsibility before using it, especially if those are not listed in this instruction manual. The manufacturer accepts no liability for damage resulting from any other use.

#### Indications

- Precision impressions for accurately fitting restorations with the highest marginal accuracy, such as
  - functional impressions
  - Situation impressions for unfavorable dental arch and jaw shapes and/or deep-set bands
- Bite registrations
- Support pin registrations

#### Contraindication

In case of known hypersensitivity to methacrylates, skin contact should be avoided.

#### Patient group

The use of Lightplast base plates is not limited to any particular patient group. Contraindications must be observed.

#### Undesirable side effects

Lightplast base plates contains methacrylates. These may cause allergic skin reactions. In rare cases sensitization is possible.

#### General description

Lightplast base plates is a pre-shaped, modelable, light-curing composite for the production of individual impression trays in the dental laboratory or dental practice. It features low shrinkage, low water absorption and high dimensional stability.

#### General safety instructions

H317: May cause an allergic skin reaction. Avoid contact of uncured material with skin/mucous membrane and eyes. It may be slightly irritating and in rare cases may cause sensitization to methacrylates. Use protective equipment: Safety glasses, suitable protective gloves with protection function to Meth-acrylates (e.g. Polyethylene-gloves, 4h-gloves). Do not wear contaminated work clothing outside the workplace.

Available variants: Lightplast base plates, plate thickness approx. 2.2 mm in the colors nature und pink.

#### User profile

Lightplast base plates is a product for professional use by dental technicians or dentist. It is intended for use in the dental laboratory or dental practice. No special training is required. The finished products made of the material (individual impression trays, base plates, support pin registrations) are intended for professional use by dentists in a dental practice. Special training is not required.

#### Clinical benefit

The clinical benefit of light-curing impression plates lies indirectly in the replacement of the dental apparatus and thus in a restoration of masticatory function, since precision and/or functional impressions are essential for the accurately fitting production of dentures.

#### Application

- The situation model should be prepared as required and insulated with an alginate insulation (e.g. Isolat film). Under-cutting areas of the model should be blocked out before.
- Open the can at the handle tab, remove one Lightplast base plates incl. protective fleece and immediately reseal the can tightly.
- The protective fleece is removed from the Lightplast base plates sheet.
- The sheet is adapted to the insulated model and trimmed at the edges.
- The handle can be formed from the excess material. The handle is not used when making a base plate.
- The sheet is then polymerized on the model in a blue light or UVA light device (e.g. EyeVolution®) for 5 minutes. The instructions for use of the light unit used must be observed!
- The sheet is removed from the model and polymerized from the back for another 3 minutes.
- The individual impression tray made in this way can be processed immediately.
- The inhibition layer (adhesive layer on the top of the material) allows subsequent correction (e.g. reinforcement of the sheet). The inhibition layer can be easily removed with a moistered tissue with an alcoholic cleaning agent.

**Composition**
Mixture of different acrylates / methacrylates, fumed silica, glass beads, photochemical initiators, vaseline, pigments

#### Warnings

Lightplast base plates is light-sensitive, so that polymerization can already be triggered by exposure to daylight. Therefore, the material should not be exposed to light unnecessarily (removal from the can only immediately before adaptation to the model, immediate sealing of the can after removal).

#### Interactions

No interactions are known.

#### Special storage instructions

Keep the can tightly closed. Store protected from light and at 4–25 °C.

#### Shelf life

3 years from manufacture. Do not use after the expiry date stated on the label!

#### Waste treatment

Do not allow product or primary packaging which has not been emptied of all residues to reach sewage system, water, subsoil or soil. Disposal according to official regulations. Non-contaminated and completely empty packaging as well as fully cured products can be disposed of with normal municipal waste (household waste). Observe waste separation.

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