Material description

Protector Adhäsiv is an air-drying one-component material that ensures the bonding between the 3D Protector and Drufosoft (EVA) material.

Intended use

Adhesive for the bonding of the dental 3D Protector (3D printed silicone) with Drufosoft (EVA).

Processing

The transparent and fast-setting adhesive is applied thinly to the backside of the 3D Protector in the area of the distal ends and the incisal edge. Allow the applied material to ventilate for about 2 minutes at room temperature.

Close the bottle immediately after use. The contact surfaces on the Drufosoft base must be clean and grease-free. If necessary, clean with IPA and allow to ventilate.

Position the 3D Protector on the Drufosoft base within max. 10 minutes (see processing instructions for the 3D Protector) and press firmly on the adhesive surfaces.

Important instructions

Keep away from children!

Only for use in the dental technique!

Contains: Ethyl acetate

Hazard statements

H225 Highly flammable liquid and vapour. H319 Causes serious eye irritation. H336 May cause drowsiness or dizziness.

Precautionary statements

P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P241 Use explosion-proof electrical / ventilating / lighting equipment. P303+ P361+P353 IF ON

Storage conditions

Store at 18°C–28°C (64°F–82°F).

Always close the bottle tightly after use!

At room temperature in the closed container.

Protect from direct sunlight.

Lot number / Expiry date

The lot number and the expiry date can be found on the outer packaging and on the bottle. In case of complaints about the product, please always state the lot number. Do not use the product after the expiry date is passed.

This product was developed for dental use and has to be applied according to the instructions. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. Furthermore, the user is bound to check the product according to suitability and possible use in self dependence before using it subject to its purpose, especially in case the purpose is not mentioned in the instructions.

Protector Adhäsiv

Gebrauchsinformation

Directions for use

SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water / shower. P305+ P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to do. Continue rinsing. P405 Store locked up. P501 Dispose of the contents / container in accordance with local / regional / national / international regulation.

Dreve Dentamid GmbH
Max-Planck-Straße 31
59423 Unna/Germany
www.dentamid.dreve.de

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