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## Intended use

Bite registration material based on vinyl polysiloxane

### Indication for use

Regifix is an addition-curing silicone for bite registrations.

### Fields of application

- Bite registrations
- Key for intraoral registration

### Contraindication

The product may not be used if the patient is allergic to one of the ingredients. There are no side-effects to be expected from this medical device if processed and used properly. However, immune reactions (e. g. allergies) or local disorders of sensation (e. g. irritation of the oral mucosa) cannot be ruled out in principle. If you become aware of adverse side-effects, also in cases of doubt, please be sure to advise us in any case and describe the circumstances and symptoms as accurately as possible. We will investigate any notice.

### Target Patient Group

Patients who need dental care.

### Intended users

Dentist

### Product description

Regifix opaque and transparent are addition-curing silicones for bite registration with extremely thixotropic consistency and high final hardness. Due to the high final hardness, Regifix opaque and transparent can be cut and milled very well.

### Processing 50 ml double cartridge

Regifix is exclusively available in the S50-double cartridge system. By means of an Injector and mixing tips an ideal 1:1 mixing of the components will be reached without losing time. Regifix can be applied cleanly and without loss to the tooth rows. Due to the thixotropy, undesired flow of Regifix into dental interstices is prevented. The application can be carried out without interruption, as the mouth temperature leads to an accelerated curing. Ensure that the bite is brought into the maximum intercuspidation within the given processing time of 30 sec. and keep it closed for further 90 sec. The fast

polymerisation evokes a slight, physiologically harmless temperature increase. After curing the registrate may be removed from the patient's mouth easily and without deformation.

Please find processing times in "technical data" overview.

### Cleaning / disinfecting

Bite registration – After removal from the mouth rinse the bite registration under running water. Disinfection can be carried out with commercially available, suitable disinfection solutions (e. g. based on quaternary ammonium compounds with benzalkonium chloride or glutaraldehyde), according to the manufacturer's instructions. Mixing canula and application tip – After use leave the mixing canula on the double cartridge, disinfect it by wiping the mixing canula with a disinfectant and do not exchange it before the next application. It serves as a sealing cap. The mixing canula and application tip are for single use only and must be replaced before the next application.

DS 50 Injector – DS 50 Injector can be used several times if properly disinfected. Disinfection can be carried out using a commercially available dipping disinfectant (e. g. glutaraldehyde) according to the manufacturer's instructions.

### Cautions

- Do not use Regifix for conventional impressions.
- The product is exclusively intended for the above-mentioned range of use and shall only be used by professionally qualified and instructed persons.
- Due to technical filling reasons there might be an unequal transportation of the two components of new cartridges. For this reason it is recommended to dispose the initially mixed pea-sized amount.
- Pre-convey the material before attaching the mixing canula.

### Batch number / use-by date

This product was developed for dental use and has to be applied according to the instructions. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. Furthermore the user is bound to check the product according to suitability and possible use in self

- Remove residues of retraction agent with water spray before taking the bite registration.
- Remove all residues of registration material from the patient's mouth.
- Polymerisation, referring to the setting of cross-linked silicones, can be disrupted by latex gloves, creams, cleaning agents etc. For this reason, we recommend that you wear commercially available polyethylene gloves.
- Wear protective clothing! Set material is chemically resistant. Marks on your clothing cannot be removed.
- After use leave the mixing canula as a cap on the double cartridge.
- Never convey to a mixing cannula which has been cured.

### Keep away from children!

### Note for one-time use

The hardened bite registration cannot be used for a further bite registration due to its physical features.

### Disposal

Disposal of contents/container in accordance with local/regional/national/international regulations and safety data sheet.

### Hazard identification

Hazard and safety notes can be found on the product label and the corresponding safety data sheet.

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dependence before using it subject to its purpose, especially in case the purpose is not mentioned in the instructions.

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