Separator

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Separator

Directions for use

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Processing:

1. Remove the material (e.g. an impression with implantation pillars) from the saliva residue and blow it dry with compressed air.
2. Spray the Separator evenly thin into the areas of the impression where the gingiva mask is planned.
3. The entire moistening!
4. The drying time amounts to approx. 1 - 2 minutes. The dried Separator shows a pale, not glossy surface.
5. When applying the second material layer please consider that the diving layer is not injured thorough instruments / mixing cannulas.
6. For further processing of the above mentioned materials please refer to the manufacturer's indications.

Important advices:

- With the Separator a separation of polyether impressions to addition-vulcanising silicones is not possible.
- Do not spray against a flame or onto glowing objects.
- Sensitive persons can show allergic reactions to one of the product components.
- The product is exclusively designed for the a. m. field of application and is only to be used by qualified and instructed persons. The general information of this product is based on detailed application practice and intensive research work. Due to the further development we reserve the right of modification.

Available in: 50 ml bottle with brush insert and spray head.
REF D4305

Contains: Hydrocarbons, C7-C9, n-Alkane, Isoalkane, Cyclene

Hazard Statements

H225 Highly flammable liquid and vapour. **H336** May cause drowsiness or dizziness. **H304** May be fatal if swallowed and enters airways. **H411** Toxic to aquatic life with long lasting effects.

Precautionary statements

P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. **P261** Avoid breathing dust/fume/gas/mist/vapours/spray. **P280** Wear protective gloves/protective clothing/eye protection/face protection. **P301+P310** IF SWALLOWED: Immediately call a POISON CENTER or doctor. **P304+P340** IF INHALED: Remove person to fresh air and keep comfortable for breathing. **P331** Do NOT induce vomiting. **EUH066** Repeated exposure may cause skin dryness or cracking.

Dreve Dentamid GmbH is not liable for any damages caused by improper application of the material.

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Die Dreve Dentamid GmbH haftet nicht für Schäden, die durch fehlerhafte Anwendung des Materials hervorgerufen werden.