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Dentist

Product description

StoneBite® scan is an addition-curing silicone for bite registration with a high final hardness. The high final hardness enables a problem-free milling or grinding. In case of breaks, clearly defined pieces will be obtained. The material is easily injectable, even through mixing canulas with a thin diameter. StoneBite® scan is reflectionactive and can be used for optical collection of data (registration of antagonists) in CAD/CAM technology.

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Intended use

Bite registration material based on vinyl polysiloxane

Indication for use

StoneBite® scan is an addition-curing silicone for bite registrations.

Fields of application

- Bite registrations

- Key for intraoral registration

Warning

StoneBite® scan should be used solely on the occlusal surface of the teeth. Do not use StoneBite® scan for conventional impressions. Due to the high end hardness the removal is difficult and may lead to inadvertent extraction of teeth.

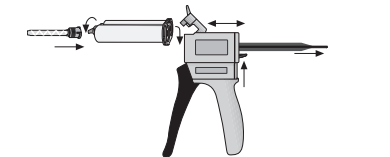
Contraindication

The product may not be used if the patient is allergic to one of the ingredients. There are no side-effects to be expected from this medical device if processed and used properly. However, immune reactions (e. g. allergies) or local disorders of sensation (e. g. irritation of the oral mucosa) cannot be ruled out in principle. If you become aware of adverse side-effects, also in cases of doubt, please be sure to advise us in any case and describe the circumstances and symptoms as accurately as possible. We will investigate any notice.

Processing 50 ml double cartridge

StoneBite® scan is exclusively available in the S50 double cartridge system. By means of an injector and mixing tips an ideal 1:1 mixing of the components will be reached without losing time. With the aid of application tips StoneBite® scan can be applied cleanly onto the teeth rows and without loss. Due to the thixotropy, undesired flow of StoneBite® scan into dental interstices is prevented. This working step is to be carried out without interruption, as the mouth temperature leads to an accelerated curing. Ensure that the bite is brought into the maximum intercuspitation within the given processing time of 30 sec. and keep it closed for further 50 sec. The fast polymerisation evokes a slight, physiologically harmless temperature increase. After curing the registrate may be removed from the patient's mouth easily and without deformation.

Please find processing times in "technical data" overview.

Cleaning / disinfection

Bite registration – After removal from the mouth rinse the bite registration under running water. Disinfection can be carried out with commercially available, suitable disinfection solutions (e. g. based on quaternary ammonium compounds with benzalkonium chloride or glutaraldehyde), according to the manufacturer's instructions.

Mixing canula and application tip – After use leave the mixing canula on the double cartridge, disinfect it by wiping the mixing canula with a disinfectant and do not exchange it before the next application. It serves as a sealing cap. The mixing canula and application tip are for single use only and must be replaced before the next application.

DS 50 Injector – DS 50 Injector can be used several times if properly disinfected. Disinfection can be carried out using a commercially available dipping disinfectant (e. g. glutaraldehyde) according to the manufacturer's instructions.

The DS 50 Injector can be used several times if properly disinfected. Disinfection can be carried out using a commercially available dipping disinfectant (e. g. glutaraldehyde) according to the manufacturer's instructions.

This product was developed for dental use and has to be applied according to the instructions. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. Furthermore the user is bound to check the product according to suitability and possible use in self dependence before using it subject

to its purpose, especially in case the purpose is not mentioned in the instructions.

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Cautions

Do not use StoneBite® scan for conventional impressions.

The product is exclusively intended for the above-mentioned range of use and shall only be used by professionally qualified and instructed persons.

Due to technical filling reasons there might be an unequal transportation of the two components of new cartridges. For this reason it is recommended to dispose the initially mixed pea-sized amount.

Pre-convey the material before attaching the mixing cannula.

Remove residues of retraction agent with water spray before taking the bite registration.

Remove all residues of registration material from the patient's mouth.

Polymerisation, referring to the setting of cross-linked silicones, can be disrupted by latex gloves, creams, cleaning agents etc. For this reason, we recommend that you wear commercially available polyethylene gloves.

Wear protective clothing! Set material is chemically resistant. Marks on your clothing cannot be removed.

After use leave the mixing canula as a cap on the double cartridge.

Never convey to a mixing cannula which has been cured.

Keep away from children!

Note for one-time use

The hardened bite registration cannot be used for a further bite registration due to its physical features.

Disposal

Disposal of contents/container in accordance with local/regional/national/international regulations and safety data sheet.

Hazard identification

Hazard and safety notes can be found on the product label and the corresponding safety data sheet.

Serious incidents

All serious incidents occurring in relation to the product shall be reported to the manufacturer and to the competent authority of the Member State in which the user and / or patient is settled.

This product was developed for dental use and has to be applied according to the instructions. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. Furthermore the user is bound to check the product according to suitability and possible use in self dependence before using it subject

to its purpose, especially in case the purpose is not mentioned in the instructions.

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