Produktbeschreibung

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Product description

Descaling agent especially for boiling out units. Hard water needs to be decalcified from time to time. Empty the unit and fill again with 10 ltr. fresh water. Add 10 % – that means 1 kg – Unisol E and heat the unit up to 50–60 °C. Start the unit and let it run until development of carbonic acid with effervescent effect (approx. 10 min.). Evacuate the decalcifying bath and rinse the unit with fresh water.

Content

Citric acid monohydrate

Hazard statements

H319 Causes serious eye irritation.

Precautionary statements

P264 Wash thoroughly after handling.

P280 Wear eye protection / face protection.

P305+P351+P338

IF IN EYES: Rinse cautiously with water for several minutes. Remove contactlenses, if present and easy to do. Continue rinsing.

P337+P313

If eye irritation persists: Get medical advice/attention.

Keep away from children!**Only for dental use/earmold lab!**

Unisol E is admitted by the Umweltbundesamt "Federal Agency for Environment" according to the Wasch- und Reinigungsmittelgesetz "guideline for washing and cleaning agents" UBA:5062 0003.

Mode of delivery Dental

REF D4383 Unisol E 2.5 kg package

Mode of delivery Otoplastik

REF 829 Unisol E 2.5 kg package

This product was developed for dental use/use in an earmold lab and has to be applied according to the instructions. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. Furthermore the user is bound to check the product according to suitability and possible use in self dependence before using it subject to its purpose, especially in case the purpose is not mentioned in the instructions.

**Achtung
Warning**