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Tandheelkundig lepelmateriaal kortstondig

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Intended use
Dental splinting material short-term

Extended purpose
Thermoforming blank and moulded tray handle for the production of individual impression-trays, base plates.

Indications for use
Drufoplast is especially suitable for the manufacture of individual trays as well as the basis for bite registrations or set-ups.

Contraindication
In case of allergies to any of the ingredients do not use Drufoplast. Not desired side effects of this medical device are not to be expected in case of proper processing and application. However, immune reactions (e. g. allergies) or irritations principally cannot be excluded. Should you get to know non-desired side effects – also in doubtful cases – we appreciate in any case to be informed with a detailed description of the attendant circumstances and symptoms. We will follow each hint.

Target patient group
Patients who need dental care

Intended users
Trained staff in dental field

Technical data
Technical datasheet available on request.

Processing
Place the plate into a vacuum or pressure unit (Drufoformat scan). Before thermoforming the plaster model should be watered and undercut areas should be blocked out with a heat resistant material (e. g. Fillin). Depending on the unit and material thickness the heating times are 1:05 to 1:50 min. With the help of vacuum or pressure the plastified plate is formed onto the model. The cooling time under pressure or vacuum should be at least as long as the heating time. Drufoplast can be treated with hard metal

cutters. In order to smooth the cuts especially flexible grinding material can be used (UltraTrim). For the manufacture of individual impression trays a handle of the same material is available. These handles can be adapted to the tray with heat and compounded with a chemical bonding.

Disinfection and reuse
A disinfection with all commercial materials is possible. If Drufoplast is used as a base plate, it can usually be reused 2 to maximum 3 times.

Note
• While working out on the material, it is recommended to use an exhaust system and protective glass or goggles as well as a face mask.
• In combination with impression silicones (e.g. Dynax®) it is recommended to use a commercially available tray adhesive.

Serious incidents
All serious incidents occurring in relation to the product shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is settled.

Storage conditions
At room temperature, i. e. 18 °C–28 °C.

Batch number / Use-by date
The lot number and the expiry date are indicated on each Drufoplast packaging. In case of claims, please always indicate the lot number of the product. Do not use the product after the expiry date.

Disposal
Disposal of contents/container in accordance with local/regional/national/international regulations.

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This product was developed for dental use and has to be applied according to the instructions. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. Furthermore the user is bound to check the product according to suitability and possible use in self

dependence before using it subject to its purpose, especially in case the purpose is not mentioned in the instructions.

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