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Intended use
Dental splinting material long-term

Indication for use
Soft thermoforming foil for the production of mouthguards, positioners and various occlusion splints.

Indication
Due to its properties, the material is suitable for the production of bite and bruxism splints, as well as for the treatment of myoarthropathies (muscle/jaw joint disorders) and resulting parafunctions such as clenching, grinding etc. It can also be used as a positioner for slight tooth movements.

Contraindication
In case of allergies to any of the ingredients do not use Drufosoft®. Not desired side effects of this medical device are not to be expected in case of proper processing and application. However, immune reactions (e. g. allergies) or irritations principally cannot be excluded. Should you get to know non-desired side effects – also in doubtful cases – we appreciate in any case to be informed with a detailed description of the attendant circumstances and symptoms. We will follow each hint.

Target patient group
Patients who need dental care.

Intended users
Trained staff in dental field

Characteristics
flexible, soft-remaining resin
Ø 120 mm / 125 mm
□ 120 mm / 125 mm

Material thickness
1.0 / 1.5 / 2.0 / 3.0 / 4.0 / 5.0 mm

Colours
nature-transparent – 3 mm blanks in various colours available

Processing
All Dreve thermoforming units are suitable for the processing of Drufosoft® blanks (Drufoformat scan, Drufoformat scan, Drufoformat, Drufoformat SQ / TE, Vacomat U / P, Vacomat 2000). For plastification the blank will be placed into the plate reception and with the help of the radiant heater brought to the necessary temperature. The times vary between 1:30 and 2:45 minutes according to the unit type and the surrounding temperature. After end of the heating

time the material hangs down → complete the thermoforming process. Before thermoforming plaster models should be watered and insulated. Depending on the material thickness the cooling time under pressure/vacuum is 3–8:30 minutes. After the cooling time Drufosoft® can be cut by means of scissors or a scalpel. Drufosoft® is suitable for “additive thermoforming”. Through process of lamination various foils can be connected (**only with pre-forming unit under pressure**).

Cleaning
For cleaning, the use of MaxiFresh™ cleaning tablets is recommended. Please be aware of the MaxiFresh™ instruction for use.

Note
While working out on the material, it is recommended to use an exhaust system and protective glass or goggles as well as a face mask.

Serious incidents
All serious incidents occurring in relation to the product shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is settled.

Storage conditions
At room temperature, i. e. 18–28°C.

Lot number / Expiry date
The lot number and the expiry date are indicated on each packaging. In case of claims, please always indicate the lot number of the product. Do not use the product after the expiry date.

Disposal
Disposal of contents/container in accordance with local/regional/national/international regulations.

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