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Intended use <p>Dental splinting material long-term</p>
Extended purpose <p>Soft thermoforming foil for the production of positioners and various occlusion splints.</p>
Indications for use <p>Due to its properties, the material is suitable for the production of bite and bruxism splints, as well as for the treatment of myoarthropathies (muscle/jaw joint disorders) and resulting parafunctions such as clenching, grinding etc. It can also be used as a positioner for slight tooth movements.</p>
Contraindication <p>In case of allergies to any of the ingredients do not use Drufosoft® pro. Not desired side effects of this medical device are not to be expected in case of proper processing and application. However, immune reactions (e. g. allergies) or irritations principally cannot be excluded. Should you get to know non-desired side effects – also in doubtful cases – we appreciate in any case to be informed with a detailed description of the attendant circumstances and symptoms. We will follow each hint.</p>
Target patient group <p>Patients who need dental care.</p>
Intended users <p>Trained staff in dental field</p>
Characteristics <p>flexible, soft-remaining resin <div> <div><div>Ø 120 mm/125 mm</div></div> <ul style="list-style-type: none">□ 120 mm/125 mm </div></p>
Material thickness <p>3.0 mm</p>
Colour <p>nature transparent</p>
Processing <p>All Dreve thermoforming units are suitable for the processing of Drufosoft® pro blanks (Drufoomat scan, DrufoSMART scan, DrufoSMART, Drufoomat SQ/TE, Vacfoomat U/P, Vacfoomat 2000). For plastification the blank will be placed into the plate reception and with the help of the radiant heater brought to the necessary temperature. The times vary between 1:45 and 2:15 minutes according to the unit type and the surrounding temperature. After end of the heating time the material hangs down → complete the thermoforming process. Before thermoforming plaster models should be watered and insulated. The cooling time under pressure/ vacuum is at least 7 minutes. After the cooling time Drufosoft® pro can be cut by means of scissors or a scalpel. Drufosoft® pro is suitable for „additive thermoforming“. Through process of lamination various foils can be connected (only with pressure forming unit).</p>
Cleaning <p>For cleaning, the use of MaxiFresh™ cleaning tablets is recommended. Please be aware of the MaxiFresh™ instruction for use.</p>
Note <p>While working out on the material, it is recommended to use an exhaust system and protective glass or goggles as well as a face mask.</p>
Serious incidents <p>All serious incidents occurring in relation to the product shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/ or patient is settled.</p>
Storage conditions <p>At room temperature, i. e. 18–28 °C</p>
Lot number / Expiry date <p>The lot number and the expiry date are indicated on each packaging. In case of claims, please always indicate the lot number of the product. Do not use the product after the expiry date.</p>
Disposal <p>Disposal of contents/container in accordance with local/regional/national/international regulations.</p>

This product was developed for dental use and has to be used according to the instructions. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. Furthermore the user is bound to check the product according to

suitability and possible use in self dependence before using it subject to its purpose, especially in case the purpose is not mentioned in the instructions.

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