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REF **D4306** **DuoSep, 10 ml**

Material description

DuoSep is a separating agent for the insulation of equal and non-compatible materials which is to be applied by means of a brush.

Field of application

A-silicone / A-silicone
C-silicone / A-silicone
Polyether / A-silicone
Polyether / polyurethane

Processing

Shake before using the material. Apply equally onto a clean and dry contact surface. Avoid the development of puddles! Let the material dry well. The dry material creates a very thin insulation layer which guarantees a safe separating film.

Important advices

- DuoSep technical filling reasons DuoSep can easily settle when storing. For this reason always shake well before use.
- DuoSep can easily be removed with water.
- For dental use only.

Storage conditions

At a room temperature, i. e. 18–28 °C
Shelf life: 2 years in the closed original container

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This product was developed for dental use and has to be used according to the instructions. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. Furthermore the user is bound to check the product according to suitability and possible use in self dependence

before using it subject to its purpose, especially in case the purpose is not mentioned in the instructions.

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