## Product description

**Finishing Liquid** is a volatile, transparent liquid for the surface treatment of Drufosoft®-thermoforming foils. **Finishing Liquid** smoothes rough surfaces of finished Drufosoft® and creates a smooth, gleaming surface.

## Processing instruction

By means of a soft kerchief (e. g. handkerchief) **Finishing Liquid** is rubbed with a light pressure onto the rough surface of Drufosoft®. Due to a chemical process the surface of Drufosoft® comes off and a rotary movement of the kerchief causes a fusing. In order to achieve a consistent gleam the total area can afterwards be reworked with a surplus of liquid and without pressure. The complete drying of **Finishing Liquid** on the treated surface needs approx. 10–15 minutes and should be effected in a dust-free area.

## Hazard statements:

H225 Highly flammable liquid and vapour. H315 Causes skin irritation. H319 Causes serious eye irritation. H317 May cause an allergic skin reaction. H336 May cause drowsiness or dizziness. H304 May be fatal if swallowed and enters airways. H411 Toxic to aquatic life with long lasting effects.

## Precautionary statements:

P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P301+P310 IF SWALLOWED: Immediately call

## Important advice

- An important component of **Finishing Liquid** has an intensive eucalyptus odour which remains for a longer period on the used Drufosoft®.
- Sensitive persons can show allergic reactions to one of the product components.
- The product is exclusively designed for the a. m. field of application and is only to be used by qualified and instructed personnel.
- The general information of this product is based on detailed application practice and intensive research work. Due to further development we reserve the right of modification.

## Storage conditions:

At room temperature, i. e. 18–28 °C

**Durability:** see date of expiry

**Contains:** Eucalyptus oil chin. 80–85 %, acetone.

## Form of delivery:

 **D4466N** **Finishing Liquid**  
250 ml bottle

a POISON CENTER / doctor. P303+P361+P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water / shower. P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P405 Store locked up. P501 Dispose of contents / container in accordance with local / regional / national / international regulations.

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