**Bedienungsanleitung • Directions for use  
Mode d'emploi • Instrucciones de uso  
Istruzioni d'uso**

# **Labormat SD Labormat TH**

**Ausbrühgerät  
Boil-out unit  
Appareil d'ébouillantage  
Aparato hervidor  
Apparecchio per ebollizione**

deutsch

english

français

español

italiano

**Inhalt** (DE)

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**Contents** (EN)

1.	Device description	38
2.	Technical data	38
3.	Commissioning	39
<b>4.</b>	<b>Declarations of conformity</b>	
4.1	Labormat SD	40
4.2	Labormat SD / USA	41
4.3	Labormat TH	42
4.4	Labormat TH / USA	43
5.	Safety instructions	44–45
<b>6.</b>	<b>Operating the Labormat SD</b>	46
6.1.	Filling with water	46
6.2.	Establishing operational readiness	46
6.3	Boiling out	47
6.4	Boiling out with the hand spray	48
6.5	Polymerising	48
7	Cleaning and maintenance	49
7.1	Checking the water level	49
7.2	Restoring operational readiness	49
7.3	Changing the water	50
7.4	Cleaning the spray tubes	51
8.	Labormat SD accessories	51
9.	Labormat SD list of spare parts	52–53
10.	Labormat SD explosion drawing	54–55
<b>11.</b>	<b>Operating the Labormat TH</b>	56
11.1	Filling with water	56
11.2	Establishing operational readiness	56
11.3	Boiling out	57
11.4	Polymerising	57
12.	Cleaning and maintenance	58
12.1	Checking the water level	58
12.2	Restoring operational readiness	58
12.3	Changing the water	58
12.4	Cleaning the spray tube	59
13.	Labormat TH list of spare parts	60–61
14.	Labormat TH explosion drawing	62
15.	Accessories (optional)	63
16.	Care products	63
17.	Troubleshooting	64–67
18.	General information	68
19.	Warranty	68
20.	Detailed Labormat SD wiring diagram	166
21.	Detailed Labormat TH wiring diagram	167

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## 1. Device description

Labormat devices are boil-out and polymerisation devices with a water tank which can be pre-heated continuously from 30–95 °C. The pump and spray tube system facilitates specific cleaning of plaster models, for example, as well as the extraction of wax components using flask technology. The freely-accessible water tank also permits temperature-controlled long-term polymerisation of hot-polymerising resins.

Both device models feature an over-heating protection function which is automatically triggered in the event of a water shortage.

### Labormat SD:

Large standalone device with an approx. capacity of 50 litres and accommodating up to 12 flasks.

### Labormat TH:

Small tabletop device with an approx. capacity of 15 litres and accommodating up to 6 flasks.

## 2. Technical data

### Labormat SD

Nominal voltage:	230 V~ / 220 V~
Nominal frequency:	50 Hz / 60 Hz
Output:	3100 W
Operating temperature:	30–95 °C
Max. water volume:	50 l
Dimensions (H x W x D):	850 x 630 x 600 mm
Weight:	75 kg

### Labormat TH

Nominal voltage:	230 V~ / 220 V~
Nominal frequency:	50 Hz / 60 Hz
Output:	3200 W
Operating temperature:	30–95 °C
Max. water volume:	15.0 l
Dimensions (H x W x D):	440 x 580 x 360 mm
Weight:	38 kg

## 3. Commissioning

**Prior to commissioning, please ensure that the mains voltage indicated on the rating plate complies with the voltage supplied by the power source.**

Establish a mains connection with the mains cable. Fill the device with water (please refer to the section „Filling with water“).

## 4. Declarations of conformity

### 4.1 Labormat SD

#### Low-Voltage Directive (2014 / 35 / EG)

Herewith we declare that the machine described below in its conception and design and in the shape delivered by us is in accordance with the fundamental requirements for safety and health as prescribed in the applicable EC directives.

In the case of any change or modification of the machine, not authorised by us, this declaration becomes invalid.

#### Denomination of the machine:

Labormat SD (D3601)

**Type of machine:** Boiling-out unit – standing model

#### The following additional EC directives were applied:

2014 / 30 / EU EMV Directive

The protection objectives of EC Directive 2014 / 35 / EU are complied with.

#### The following harmonized standards have been applied:

##### 2014 / 35 / EU

##### Low Voltage Directive

EN 61010-1:2010 Safety requirements for electrical equipment for measurement, control and laboratory use – Part 1: General requirements (IEC 61010-1:2010 + Cor.:2011); German version EN 61010-1:2010.

EN 61010-2-010:2014 Safety requirements for electrical equipment for measurement, control, and laboratory use - Part 2-010: Particular requirements for laboratory equipment for the heating of materials (IEC 61010-2-010:2014); German version EN 61010-2-010:2014.

##### 2014 / 30 / EU

##### EMC Directive

EN 61000-3-2:2014 Electromagnetic compatibility (EMC) – Part 3-2: Limits – Limits for harmonic current emissions (equipment input current  $\leq$  16 A per phase) (IEC 61000-3-2:2014); German version EN 61000-3-2:2014.

EN 61000-3-3:2013 Electromagnetic compatibility (EMC) – Part 3-3: Limits – Limitation of voltage changes, voltage fluctuations and flicker in public low-voltage supply systems, for equipment with rated current  $\leq$  16 A per phase and not subject to conditional connection (IEC 61000-3-3:2013); German version EN 61000-3-3:2013.

EN 61326:1:2013 Electrical equipment for measurement, control and laboratory use – EMC requirements – Part 1: General requirements (IEC 61326-1:2012); German version EN 61326-1:2013 Emission according to living area, business and industrial undertakings as well as small enterprises interference stability according to industrial area.

EN 55014-1:2006  
A1:2009 + A2:2011 Measurement of interference voltage according to EN55014-1: 2006 frequency range 150kHz–30MHz Conducted interference emission, particular evaluation of click noise interferences – Part 1: Emission (CISPR 14-1:2005 + A1:2008 + Cor.:2009 + A2:2011); German version EN 55014-1:2006 + A1:2009 + A2:2011.

### 4.2 Labormat SD, USA

#### Low Voltage Directive (2014 / 35 / EU)

Herewith we declare that the machine described below in its conception and design and in the shape delivered by us is in accordance with the fundamental requirements for safety and health as prescribed in the applicable EC directives.

In the case of any change or modification of the machine, not authorised by us, this declaration becomes invalid.

#### Denomination of the machine:

Labormat SD (D3601USA)

**Type of machine:** Boiling-out unit – standing model

#### The following additional EC directives were applied:

2014 / 30 / EU EMV Directive

The protection objectives of EC Directive 2014 / 35 / EU are complied with.

#### The following harmonized standards have been applied:

##### 2014 / 35 / EU

##### Low-Voltage Directive

EN 61010-1:2010 Safety requirements for electrical equipment for measurement, control and laboratory use – Part 1: General requirements (IEC 61010-1:2010 + Cor.:2011); German version EN 61010-1:2010.

EN 61010-2-010:2014 Safety requirements for electrical equipment for measurement, control, and laboratory use - Part 2-010: Particular requirements for laboratory equipment for the heating of materials (IEC 61010-2-010:2014); German version EN 61010-2-010:2014.

##### 2014 / 30 / EU

##### EMC Directive

EN 61000-3-2:2014 Electromagnetic compatibility (EMC) – Part 3-2: Limits – Limits for harmonic current emissions (equipment input current  $\leq$  16 A per phase) (IEC 61000-3-2:2014); German version EN 61000-3-2:2014.

EN 61000-3-3:2013 Electromagnetic compatibility (EMC) – Part 3-3: Limits - Limitation of voltage changes, voltage fluctuations and flicker in public low-voltage supply systems, for equipment with rated current  $\leq$  16 A per phase and not subject to conditional connection (IEC 61000-3-3:2013); German version EN 61000-3-3:2013.

EN 61326:1:2013 Electrical equipment for measurement, control and laboratory use – EMC requirements – Part 1: General requirements (IEC 61326-1:2012); German version EN 61326-1:2013 Emission according to living area, business and industrial undertakings as well as small enterprises interference stability according to industrial area.

EN 55014-1:2006  
A1:2009 + A2:2011 Measurement of interference voltage according to EN55014-1: 2006 frequency range 150kHz–30MHz Conducted interference emission, particular evaluation of click noise interferences – Part 1: Emission (CISPR 14-1:2005 + A1:2008 + Cor.:2009 + A2:2011); German version EN 55014-1:2006 + A1:2009 + A2:2011.

## 4.3 Labormat TH

### Low Voltage Directive (2014 / 35 / EU)

Herewith we declare that the machine described below in its conception and design and in the shape delivered by us is in accordance with the fundamental requirements for safety and health as prescribed in the applicable EC directives.

In the case of any change or modification of the machine, not authorised by us, this declaration becomes invalid.

### Denomination of the machine:

Labormat TH (D3603N)

**Type of machine:** Boiling-out unit – standing model

### The following additional EC directives were applied:

2014 / 30 / EU EMV Directive

The protection objectives of EC Directive 2014 / 35 / EU are complied with.

### The following harmonized standards have been applied:

#### 2014 / 35 / EU

#### Low-Voltage Directive

EN 61010-1:2010 Safety requirements for electrical equipment for measurement, control and laboratory use – Part 1: General requirements (IEC 61010-1:2010 + Cor.:2011); German version EN 61010-1:2010.

EN 61010-2-010:2014 Safety requirements for electrical equipment for measurement, control, and laboratory use – Part 2-010: Particular requirements for laboratory equipment for the heating of materials (IEC 61010-2-010:2014); German version EN 61010-2-010:2014.

#### 2014 / 30 / EU

#### EMC Directive

EN 61000-3-2:2014 Electromagnetic compatibility (EMC) – Part 3-2: Limits – Limits for harmonic current emissions (equipment input current  $\leq$  16 A per phase) (IEC 61000-3-2:2014); German version EN 61000-3-2:2014.

EN 61000-3-3:2013 Electromagnetic compatibility (EMC) – Part 3-3: Limits – Limitation of voltage changes, voltage fluctuations and flicker in public low-voltage supply systems, for equipment with rated current  $\leq$  16 A per phase and not subject to conditional connection (IEC 61000-3-3:2013); German version EN 61000-3-3:2013.

EN 61326:1:2013 Electrical equipment for measurement, control and laboratory use – EMC requirements – Part 1: General requirements (IEC 61326-1:2012); German version EN 61326-1:2013 Emission according to living area, business and industrial undertakings as well as small enterprises interference stability according to industrial area.

EN 55014-1:2006  
A1:2009 + A2:2011 Measurement of interference voltage according to EN55014-1:2006 frequency range 150kHz–30MHz Conducted interference emission, particular evaluation of click noise interferences – Part 1: Emission (CISPR 14-1:2005 + A1:2008 + Cor.:2009 + A2:2011); German version EN 55014-1:2006 + A1:2009 + A2:2011.

## 4.4 Labormat TH / USA

### Low Voltage Directive (2014 / 35 / EU)

Herewith we declare that the machine described below in its conception and design and in the shape delivered by us is in accordance with the fundamental requirements for safety and health as prescribed in the applicable EC directives.

In the case of any change or modification of the machine, not authorised by us, this declaration becomes invalid.

### Machine designation:

Labormat TH (D3603NUSA)

**Type of machine:** Boiling-out unit – standing model

### The following additional EC directives were applied:

2014 / 30 / EU EMV Directive

The protection objectives of EC Directive 2014 / 35 / EU are complied with.

### The following harmonized standards have been applied:

#### 2014 / 35 / EU

#### Low-Voltage Directive

EN 61010-1:2010 Safety requirements for electrical equipment for measurement, control and laboratory use – Part 1: General requirements (IEC 61010-1:2010 + Cor.:2011); German version EN 61010-1:2010.

EN 61010-2-010:2014 Safety requirements for electrical equipment for measurement, control, and laboratory use – Part 2-010: Particular requirements for laboratory equipment for the heating of materials (IEC 61010-2-010:2014); German version EN 61010-2-010:2014.

#### 2014 / 30 / EU

#### EMC Directive

EN 61000-3-2:2014 Electromagnetic compatibility (EMC) – Part 3-2: Limits – Limits for harmonic current emissions (equipment input current  $\leq$  16 A per phase) (IEC 61000-3-2:2014); German version EN 61000-3-2:2014.

EN 61000-3-3:2013 Electromagnetic compatibility (EMC) – Part 3-3: Limits – Limitation of voltage changes, voltage fluctuations and flicker in public low-voltage supply systems, for equipment with rated current  $\leq$  16 A per phase and not subject to conditional connection (IEC 61000-3-3:2013); German version EN 61000-3-3:2013.

EN 61326:1:2013 Electrical equipment for measurement, control and laboratory use – EMC requirements – Part 1: General requirements (IEC 61326-1:2012); German version EN 61326-1:2013 Emission according to living area, business and industrial undertakings as well as small enterprises interference stability according to industrial area.

EN 55014-1:2006  
A1:2009 + A2:2011 Measurement of interference voltage according to EN55014-1:2006 frequency range 150kHz–30MHz Conducted interference emission, particular evaluation of click noise interferences – Part 1: Emission (CISPR 14-1:2005 + A1:2008 + Cor.:2009 + A2:2011); German version EN 55014-1:2006 + A1:2009 + A2:2011.

## 5. Safety instructions

**Attention!** Please read these instructions carefully before connecting and commissioning the device. The operational safety and function of the device can only be guaranteed if both the general statutory safety and accident prevention guidelines and the safety information in the operating instructions are observed.

1. The device may only be used in accordance with these operating instructions. We do not accept any liability for damage incurred as a result of non-designated use or incorrect operation.
2. The voltage indicated on the rating plate must comply with the voltage supplied by the power source.
3. The device may only be connected to a power outlet with a protective conductor. Never touch the mains plug with wet hands.
4. The power supply for Labormat devices should only be established by a separately-fused power supply line.
5. The device must be set up on a stable and level surface.
6. The heating may only be switched on and operated when the device is filled with a sufficient volume of water.
7. The condition and operational safety of the device must always be examined before use. If the device is not in perfect condition, it may no longer be used and must be marked accordingly.
8. Do not lock the device operating elements into place.
9. The timer operating elements should not be actively turned anticlockwise. This overstresses the internal spring causing the timer to wear faster.
10. When working for longer periods of time at the maximum operating temperature, the Labormat lid and interior parts of the housing can heat up to 80 °C; do not therefore touch them even briefly except using the designated handles.
11. Do not lean over the device when opening the lid. Evaporation by hot steam represents a risk of scalding.
12. Signs and labels must be kept clearly legible and may not be removed.
13. The device should only be cleaned and maintained or parts removed when the unit has cooled down. The mains plug must be pulled.
14. Waste water should only be discharged into the sewerage system at temperatures below 75 °C. Higher temperatures can damage the waste water lines over the long term.

15. The device may only be opened and repaired by authorised specialists.

16. Only accessories and spare parts approved by the manufacturer may be used. We do not accept any liability for damage incurred by using parts which have not been approved.

As we constantly strive to develop our products, we reserve the right to make technical modifications.

## 6. Operating the Labormat SD

### 6.1 Filling with water

Open the lid of the device and remove the flask baskets (optional) and support plates (55). The water inlet nozzle is located in the middle of the back wall of the water tank. The enlarged lower end of the nozzle marks the maximum permissible water level.

**Important: Never fill the device above this mark. Overfilling can cause leaks in the pump system which in turn causes water to be discharged.**

There are two ways of filling the device:

#### 1. Manual filling

Use a suitable vessel (e. g. bucket) or water hose to fill the requisite water

volume (approx. 50 litres) directly into the heating basin through the opening in the device.

#### 2. Fixed connection

The pressure hose supplied connects the Labormat SD directly to the water line (shut-off valve required). Unscrew the seal nut (39) on the water inlet and connect the pressure hose securely to the lock nut. Connect the other end of the connection hose to the water line directly via a shut-off valve. Open the shut-off valve and fill in water as far as the maximum level. Replace the support plates (55) and flask baskets in the device and close the lid (3).

### 6.2 Establishing operational readiness

1. Press the red main switch (14) in the control panel to activate the power supply. The pilot light integrated in the switch lights up indicating operational readiness of the device.

2. Set the rotary knob (19) on the thermostat (24) to the requisite water temperature.

**Important: The Labormat SD pump and spray function can only be operated as of an operating temperature of 65 °C which is indicated**

**by the green pilot lamp in the control panel (13).**

This reduces the risk of functional defects caused by wax deposits in the pump housing and the spray system. When the pre-set temperature is reached, the yellow pilot light (17) in the control panel stays on.

When the water is at the maximum level, it takes approx. 70–80 minutes to heat up to 65 °C; it takes 120–130 minutes to heat up to 95 °C.

In order to avoid delays caused by the heating phase, the Labormat SD is fitted with a 24-hour timer (33). When the power supply is activated (main switch (14) lights up red), the timer is set to continue as planned. When the white pilot light (16) in the control panel lights up, the spray and heating function is deactivated. After the

### 6.3 Boiling out

The two lids (3) are sliding doors which open in opposite directions with the result that only one side of the device can ever be opened at a time. Use the handle to slide one of the lids inwards and place the items to be boiled out on the support plate (55) or into the flask baskets available as an extra option. Close the lid (3) and activate the timer (18) in the control panel. The spray function can be pre-selected variably between 0 and 6 minutes. As an alternative to the timer, the green

specified lead time, the Labormat SD automatically heats up to the selected temperature.

**Calculation of the time period that has to be set:**

Number of hours between the set time and the planned use minus heating time (depending on set temperature).

hold button (15) in the control panel can also be pressed to automatically trigger the spray process which is maintained for as long as the hold button is pressed.

**Important: Spray tubes, flask baskets and all other internal metal surfaces display high temperatures during and after use. They should not be touched – even briefly – without using the designated handles**

## 6.4 Boiling out with the hand spray

The Labormat SD avails of double lids. The item to be cleaned (e. g. articulator) can be placed on the perforated plate covering for brief boiling out using the hand spray (30). The spray water is redirected into the water tank via discharge holes in the inside lid. When using the hand spray (30), the change-over lever (52) in the control panel must be rotated right to set it at the „hand spray“ position. Remove the hand spray (30) from the holder, set the timer (18) to the desig-

nated cleaning time or press the hold button (15) while pressing the pressure switch on the hand spray. Return to normal spray mode by rotating the change-over lever (52) to the left to set it at the „spray“ position.

**Important: The timers on the Labormat device should not be turned anticlockwise mechanically. This over-stresses the internal parts and can expose the clocks to fast wear.**

## 6.5 Polymerising

Labormat devices are also suitable for polymerising hot-polymerising resins using the metal flask technology.

Do this by opening the lid of the device (3), removing the flask baskets and support plates (55) and submerging the flasks directly in the preheated water tank.

**Caution: risk of burning – only touch using the designated handles.**

It must be ensured that flasks are not placed on the heating coil and measur-

ing sensors in the front section of the water tank. This can cause functional defects or damage.

Various temperature levels can be realised via the thermostat controller (24) to ensure homogeneous resin polymerisation.

During the polymerisation process, the Labormat can also be used as a boil-out unit; replace the support plates and flask baskets in the device and proceed as outlined in the section on „Boiling out“.

## 7. Cleaning and maintenance

### 7.1 Checking the water level

Labormat devices are hot water devices which release steam during the spray process and when the lid is open. This lowers the water level and necessitates checking the water level on a daily basis. When there is a gap of approx. 10 cm between the water surface and the maximum water level mark,

it is imperative that water is added (please refer to the section on „Filling with water“).

If the water level falls to the level of the heating coil (29), a safety switch isolates the entire device from the mains supply to protect against overheating.

### 7.2 Restoring operational readiness after triggering the safety switch

**Important: Always pull the main plug before opening the device.**

The overheating protection safety switch (21) is located on the control panel (9) and is secured with a protective cap.

If reactivation is necessary, release the protective cap in an anticlockwise

direction and press the green locking button underneath until it audibly clicks into place.

Immediately replace the protective cap on the locking button on account of the risk of electrical components being damaged by water.

### 7.3 Changing the water

In order to obtain consistently good results when cleaning and boiling out models, the wax dissolved in water should be removed regularly (once a week, more often if used more frequently). Do this by setting the temperature controller (24) to 0 °C or switching off the device and allowing it to cool (ideally overnight).

The wax dissolved in the water automatically collects as a layer on the surface. After removing the flask baskets and support plates (55), this layer can be scooped out along with any other impurities floating on the surface. When giving the device an overall and thorough cleaning (min. once a week, more frequently if used more often), the device can be switched back on again after scooping out the wax layer and preheated to 65 °C.

The water inside the device can be pumped off using the discharge hose supplied. Do this by removing a spray tube (47) (see next section) and establishing the tube/Labormat SD connection with the aid of the bayonet coupling attached to the tube. Place the other end of the tube in a drain (e. g. sink).

Activate the pump with the aid of the timer (18) or hold button (15). The water is automatically pumped off until 3–4 cm sediment remains on the base. The heavily-soiled residual water is not pumped off but rather discharged via a drainage valve (42) located on the base of the water tank. Fully open the lower front cover (5) (magnetic catch) and drainage valve (42) using the red shut-off valve. The residual water flows off and is collected in the collection basin (54) underneath. Remove rough deposits and deposits adhering to the inside of the water tank and rinse the device with a little water. Close the drainage valve (42). Detach the collection basin (54) from the device and dispose of the waste water. Fill with fresh water as far as the maximum mark outlined in the section on „Filling with water“.

**Important: Only dissolve Unisol W in cold water; high water temperatures cause foaming.**

### 7.4 Cleaning the spray tubes


Pump mode can also enable rough impurities to penetrate the spray tubes (47) along with the water. This can impair the function of the spray nozzles and should be checked at regular intervals, cleaning if necessary. This necessitates removal of the spray tubes (47) which are connected to the water line in the device via bayonet catches. Unlock the bayonet catch by a half-turn to the right and gently pull the entire spray tube (47) out at an upward incline. A cleaning brush is supplied with each new device. Clean

by releasing the dummy plug (48) at the end of the tube, inserting the brush into the spray tube (47) and use rotary and thrusting movements to release and remove rough dirt and calcium deposits. The tube should then be rinsed under running water.


The spray tubes are reassembled in the reverse order to the process described above.

**Important: When inserting the spray tubes, ensure that the spray nozzles face downwards.**


## 8. Labormat SD accessories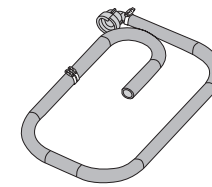

Waste water discharge hose  
REF D51142KPL

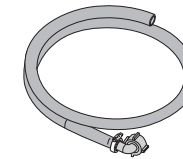Fresh water supply hose  
REF D1017E


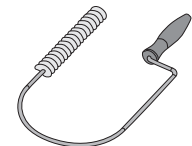
Cleaning brush for spray tubes  
REF 51074

## 9. Labormat SD list of spare parts

No.	Designation	REF
<b>Version 230 V~ / 50 Hz</b>		
1	Labormat SD housing	50572FT
3	Set Sliding lid	50765
5	Cover	none
9	Front cover	50642BD
10	Hand spray holder with nut	51347
12	Submersible pump 230 V / 50 Hz	50703
13	Green pilot light	51057
14	Red rocker switch with PVC cap	50820
15	Green push buttons with PVC cap	50843
16	White pilot light	50868
17	Yellow pilot light	50856
18	6-min timer incl. fixing parts	D6550851
19	Rotary knob	50865
20	Scale ring 0–6 min	50862
21	Thermostat EMF-80/b3/Boil-dry protection	50758
22	Scale ring temp. 30–100 °C	50901
23	Capacitor 10µF	50798
24	Thermostat 718RU-5880	D50772A2
25	Mini contactor 4S 230 V / 50 Hz	D608052
26	Perbunan seal 130-102 4-hole	50871
29	Tubular heating element	50784
30	Labormat SD hand spray incl. hose	D65029SD
31	Temperature monitor R28	50723
33	24-hour timer 230 V / 50 Hz incl. fixing parts	D6550850
34	Scale ring 0–24 h	50864
35	¾" x 10 extension	50872
38	Vaillant seal	51150
39	¾" cap	50878
40	Magnetic catch	50870
41	Connection line	51025
42	¾" x ¾" slide valve	51181
44	KAFC seal 27 x 32 x 2.5	51140
45	Brass quick coupling	50272
46	Spare Viton seal	51059
47	Spray tube incl. Nr. 48 + 2 x Nr. 46	D65012
48	Neoprene plug	51195
49	Galvanised castor	50194

No.	Designation	REF
50	Handle Labormat SD	50284
52	Ball valve 63 mm cpl. with knob bar	D65014
54	Grey box / Water collection basin	50430
55	Flask support plate	50418
56	Black cover ring	50212
<b>Version 220 V~ / 60 Hz</b>		
12	Submersible pump 220 V / 60 Hz, US version	5070360
25	Mini contactor 4S 220 V / 60 Hz, US version	D60805260
33	24-hour timer 220 V / 60 Hz, US version incl. fixing parts	D6550827
41	Mains connector with cable, US version	50694

## 10. Labormat SD explosion drawing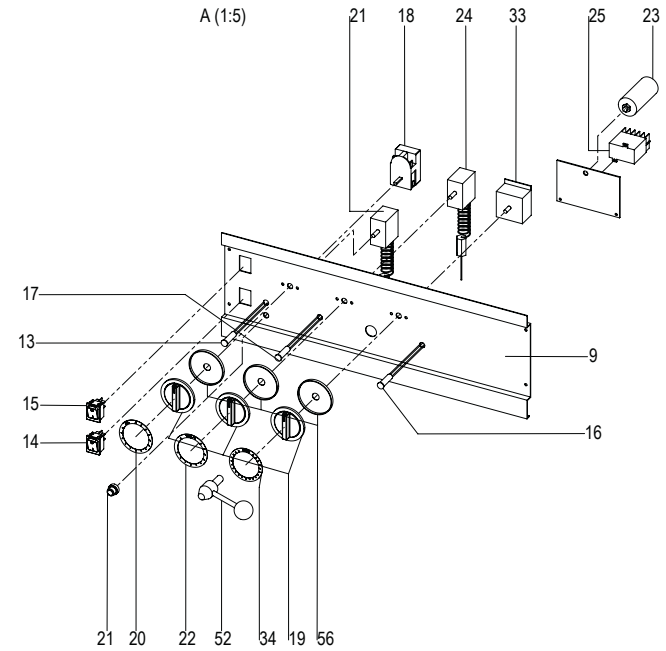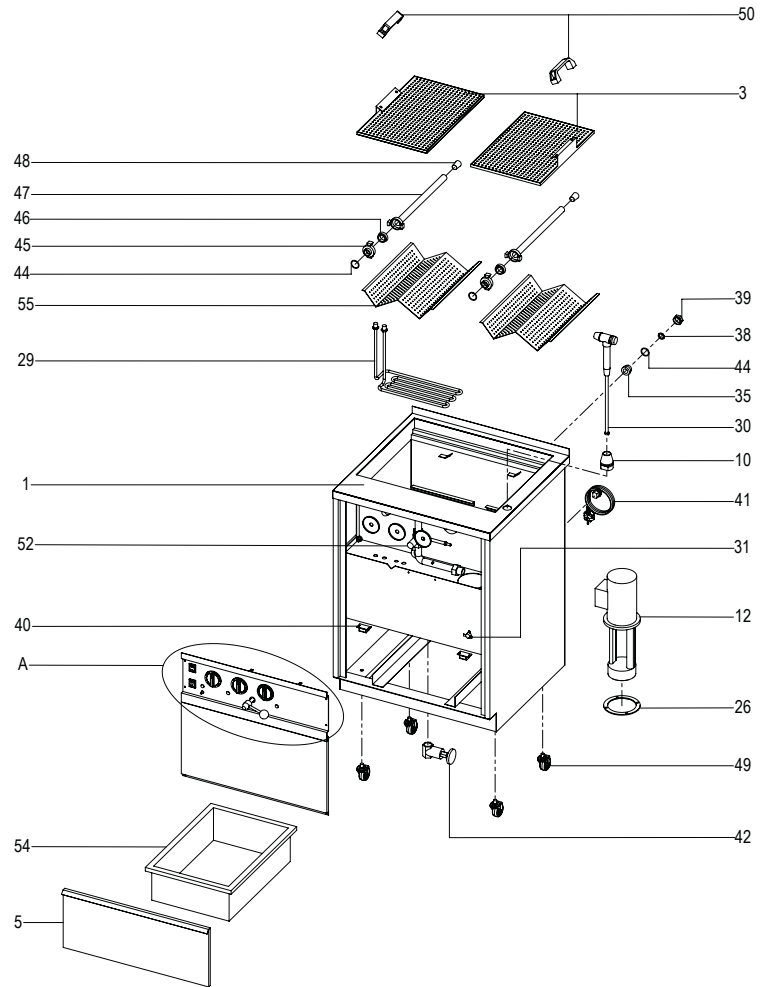

## 11. Operating the Labormat TH

### 11.1 Filling with water

Open the lid of the device (4) and remove the flask baskets (optional) and support plate (55). The rear wall of the water tank features an angle mark. This indicates the maximum permissible water level.

**Important: Never fill the device above this mark. Overfilling can cause leaks in the pump system which in turn causes water to be discharged.**

There are two ways of filling the device:

#### 1. Manual filling

Use a suitable vessel (e. g. bucket) or water hose to fill the requisite water volume (approx. 15 litres) directly into the heating basin through the opening in the device.

#### 2. Fixed connection

A water supply hose (REF D1017E) available as an option connects the Labormat TH directly with the water line (shut-off valve required). Unscrew the seal nut at the back of the device and firmly connect the supply hose with the lock nut on the water inlet (top left on rear of device). Connect the other end of the connection hose to the water line directly via a shut-off valve. Open the shut-off valve and fill in water as far as the maximum level. Replace the support plate (55) and flask baskets (accessories, please refer to page 31) in the device and close the lid (4).

### 11.2 Establishing operational readiness

1. Press the red main switch (14) in the control panel (7) to activate the power supply. The pilot light integrated in the switch lights up indicating operational readiness of the device.

2. Set the rotary knob (19) on the thermostat (22) to the requisite water temperature. **Important: The Labormat TH pump does not work until an operating temperature of 65 °C has been reached.** This reliably prevents wax deposits in the pump housing and spray system thereby preventing func-

tional faults. When the pre-set temperature is reached, the yellow pilot light (17) in the control panel stays on.

It takes approx. 20 minutes to heat the water to 65 °C.

It takes approx. 40 minutes to heat the water to 95 °C.

### 11.3 Boiling out

Raise the double lid (4) at the handle (53) and fold backwards. Place the items to be boiled out on the support plate (55) or in the flask baskets available as an option. Close the lid (4) and activate the timer (19/20) in the control panel (7). The spray function can be pre-selected variably between 0 and 6 minutes. As an alternative to the timer (19/20), the green hold button (15) in the control panel can also be pressed to automatically trigger the spray process which is maintained for as long as the hold button is pressed.

**Important: The timers on the Labormat device should not be turned anticlockwise mechanically. This over-stresses the internal parts and can expose the clocks to fast wear.**

The Labormat TH avails of a double lid. For brief boiling out using the hand spray (30), only raise the sealing cover at the handle (53) and fold backwards. Place the item to be cleaned (e. g. articulator) on the perforated metal lid and remove the hand spray (30) from the holder (10).

Set the timer (19/20) to the specified cleaning time or press the hold button (15) while pressing the pressure switch on the hand spray.

The white pilot light (16) lights up in the control panel during the active spray process.

**Important: Spray tubes, flask baskets and all other internal metal surfaces display high temperatures during and after use. They should not be touched – even briefly – without using the designated handles.**

### 11.4 Polymerising

Labormat devices are also suitable for polymerising hot-polymerising resins using metal the flask technology. Do this by opening the double lid (4) and removing the flask baskets (optional) and support plate (55) (**Caution: Risk of burning – only touch using the designated handles**) and submerge the flasks directly in the preheated water tank. The base of the water tank features a perforated metal cover (6) which protects the heating element (28) underneath and the sensors for the

thermostat (24) and boil-dry protection (21), and must not be removed or displaced except when subjecting the device to overall cleaning. Various temperature levels can also be realised via the thermostat controller (19/22) to ensure homogeneous plastic polymerisation. During the polymerisation process, the Labormat can also be used as a boil-out unit; replace the support plate (55) and flask baskets (optional) in the device and proceed as outlined in the section on „Boiling out“.

## 12. Cleaning and maintenance

### 12.1 Checking the water level

Labormat devices are hot water devices which release steam during the spray process and when the lid is open. This lowers the water level and necessitates checking the water level on a daily basis.

**If the water level reaches the level of the heating cover (6), it is imper-**

**ative that water is added** (Please refer to the section „Filling with water“). If the water level falls to the level of the heating element (28), a safety switch (21) isolates the entire device from the mains supply to protect against overheating.

### 12.2 Restoring operational readiness

**Important: Always pull the mains plug before opening the device.**

The overheating protection safety switch (21) is located on the control panel (7) and is secured with a protective cap. If reactivation is necessary, release the protective cap in an anticlockwise

direction and press the green locking button underneath until it audibly clicks into place.

Immediately replace the protective cap on the locking button on account of the risk of electrical components being damaged by water.

### 12.3 Changing the water

In order to obtain consistently good results when cleaning and boiling out models, the wax dissolved in water should be removed daily.

Do this by setting the temperature controller (19/22) to 0°C or switching off the device and allowing it to cool (ideally overnight).

The wax dissolved in the water automatically collects as a layer on the surface as the water cools down. After removing the flask baskets and support plate (55), this layer can be scooped out along with any other floating

impurities. Give the device an overall and thorough cleaning at least once a week and more frequently if used more often after scooping out the wax layer. Discharge the residual water through the drainage valve (43) on the bottom left of the rear wall. The valve (43) is opened by turning the red regulating wheel to the left on the left housing wall. The waste water can be collected in a bucket or directed into the sewerage system through a hose (only possible with the corresponding incline). Close the drainage valve (43) and

remove any rough impurities from the water tank. Fill with fresh water as far as the maximum mark outlined in the section on „Filling with water“.

We recommend adding Unisol W (see „Accessories“) for improved wax solu-

bility and avoiding emulsification of the water with residual wax.

**Important: Only dissolve Unisol W in cold water; high water temperatures cause foaming.**

### 12.4 Cleaning the spray tube

Pump mode can also enable rough impurities to penetrate the spray tube (47) along with the water. This can impair the function of the spray nozzles and should be checked at regular intervals, cleaning if necessary.

This necessitates removal of the spray tube (47) which is connected to the water line in the device via a bayonet catch (45). Unlock the bayonet catch by a half-turn backwards and gently pull the entire spray tube out at a forward/upward incline. A cleaning brush is supplied with each new device.

Clean by removing the dummy plug (48) at the end of the tube, inserting the brush into the spray tube and use rotary and thrusting movements to release and remove rough dirt and calcium deposits. The tube should then be rinsed under running water.

The spray tube is reassembled in the reverse order to the process described above.



**Important: When inserting the spray tube, ensure that the spray nozzles face downwards.**

## 13. Spare part list Labormat TH

No.	Designation	REF
<b>Version 230 V~ / 50 Hz</b>		
2	Labormat TH housing	50571TH
4	Lid set incl. handles	D50717
6	Heating cover	ohne
7	Front cover	D6550628
8	Side cover	ohne
10	Hand spray support with nut	51347
14	Red rocker switch with PVC cap	50820
15	Green push buttons with PVC cap	50843
16	White pilot light	50868
17	Yellow pilot light	50856
18	6-min timer incl. fixing parts	D6550851
19	Rotary knob	50865
20	Scale ring 0–6 min	50862
21	Thermostat EMF-80/b3/Boil-dry protection	50758
22	Scale ring temp. 30–100 °C	50901
23	Capacitor 10µF	50798
24	Thermostat 718RU-5880	D50772A2
25	Mini contactor 4S 230 V / 50 Hz	D608052
26	Perbunan seal 130-102 4-hole	50871
27	Submersible pump TB100-120 230 V / 50 Hz	50707
28	Tubular heating element	50785
30	Labormat TH hand spray incl. hose	D65029TH
31	Temperature monitor R28	50723
32	Ventilation rosette	50811
36	10 x 13.5 x 1.5 CU seal	53010
37	Red brass reducing nipple, brass with 2 x 1/2" external threads	51177
41	Connection line	51025
43	1/2" x 1/2" internal sleeve slider	51147
44	KAFC seal 27 x 32 x 2,5	51140
45	Brass quick coupling	50272
46	Spare Viton seal	51059
47	Spray tube incl. No. 48 + 2 x No. 46	D65012
48	Neoprene plug	51195
51	Rubber foot 31/17.5/6-SW	51048
53	Handle Labormat TH	D50346
55	Flask support panel	50418
56	Cover ring, black	50212


No.	Designation	REF
<b>Version 220 V~ / 60 Hz</b>		
25	Mini contactor 4S 220 V / 60 Hz, US version	D60805260
27	Submersible pump TB100-120 220 V / 60 Hz, US version	5070760
41	Mains plug with cable, US version	50694

## 14. Explosion drawing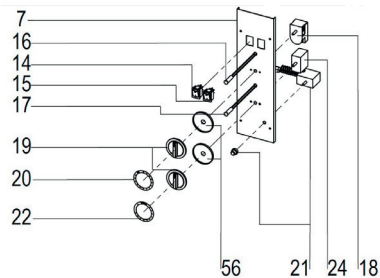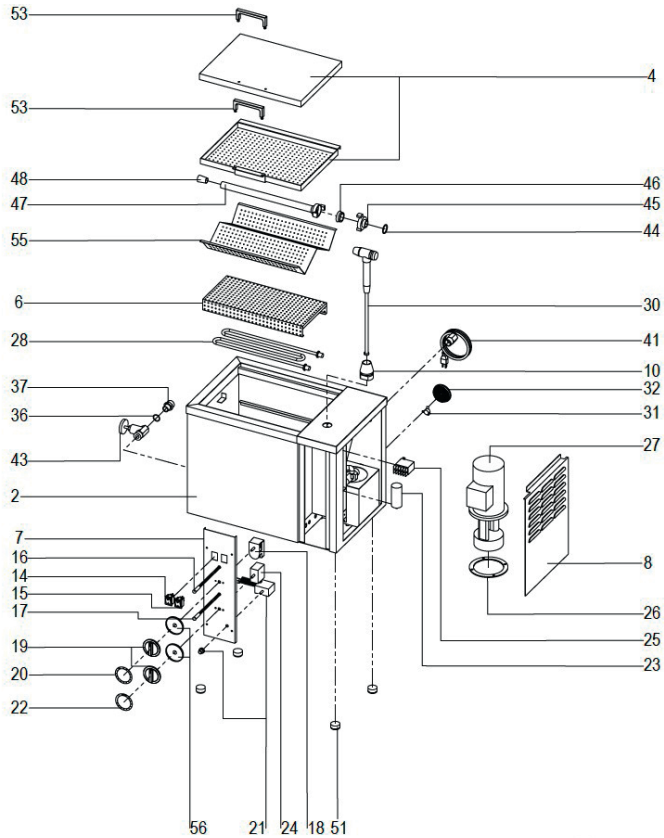

## 15. Accessories (optional)

**Flask basket** (REF D3604),  
for accommodating 3 flasks; stain-  
less steel with heat-insulating plastic  
handle  
Holding capacity:  
Labormat TH 2 pcs.  
Labormat SD 4 pcs

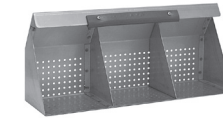**Frame** (REF D3607)  
for Labormat TH  
Water-resistant coated frame on  
castors for mobile applications  
H x W x D = 435 x 590 x 370 mm

**Filter mats** (REF D36018)  
10 pcs.  
Water-permeable fabric for lining  
between the support panel and flask  
baskets for pre-cleaning used water

## 16. Care products

**Unisol W**  
2 x 1 litre (REF D4392),  
5 litres (REF D4393)  
Wax-soluble additive for boil-out units

**Unisol E**  
1 x 2.5 kg (REF D4383), descaler for  
boil-out units

## 17. Troubleshooting

Fault	Cause	Remedy	Fault	Cause	Remedy
Device shows no function	Connection to the power supply	Establish the plug connection, replacing the plug if necessary	Relay/Contactor is defective	Relay/Contactor is defective	Examine the relay/contactor, replacing if necessary
	Defective connection cable	Examine the connection cable, replacing if necessary		Heating is defective	Examine the heating, replacing if necessary
	Main switch	Examine the main switch, replacing if necessary		Safety switch is triggered	Activate the safety switch (see instructions for activation)
	Mains network	Check mains network/output, 230 V~/ 50 Hz or 220 V~/ 60 Hz, adapting if necessary. Check fuse 16 A fuse	Device does not heat properly; temperature is wrong	Thermostat is defective or set incorrectly	Examine the thermostat, readjusting or replacing if necessary
Main switch lights up, device without function	24-hour timer is in operation	Wait until the clock expires	Relay / Contactor is defective	Examine the relay/contactor, replacing if necessary	
	Internal cabling interrupted	Check cables, terminals, cable lugs + connections. Remove cover	Device makes noises	Thermostat is defective	Examine the thermostat, replacing if necessary (remove cover to replace defective parts)
	Safety switch is triggered	Activate safety switch See instructions for activating	Relay / Contactor is defective	Examine the relay/contactor, replacing if necessary	
Heating without function	Thermostat setting	Set the desired temperature on the thermostat	Compensating capacitor is defective	Replace compensating capacitor	
	Thermostat is defective	Examine the thermostat, replacing if necessary; remove cover to replace defective part	Pump without function	Water temperature of 65 °C not reached yet	Wait until temperature level of 65 °C is reached
				Timer is defective or blocked	Examine the timer, replacing if necessary

# Labormat SD, TH

Fault	Cause	Remedy
Pump without function	Pump is blocked or defective	Examine the pump, replacing if necessary
	Boil-dry protection is triggered	Activate the boil-dry protection function (see instructions for activation)
The hold button does not react	Hold button is defective	Replace the hold button
	Water temperature of 65 °C not reached yet	Wait until temperature level of 65 °C is reached
6-min timer without return function, pump running permanently	6-min timer is blocked	Eliminate blockage, re-centering the switch if necessary
	6-min timer is defective	Examine the timer, replacing if necessary

## Only Labormat TH (REF D3603N / D3603NUSA)

Fault	Cause	Remedy
Pump is leaky	Water level is too high in the tank	Reduce water to max. level, observe marking, max. 15 litres
	Four-hole seal (between pump and basin) is defective	Dismantle pump and replace four-hole seal

## Only Labormat SD (REF D3601 / D3601USA)

Fault	Cause	Remedy
24-h timer without / with limited return function	24-h timer is defective	Compare the power source with the voltage supply indicated, adapting if necessary 230 V~/ 50Hz 220 V~/ 60Hz
	24-h timer is defective	Eliminate blockage, re-centering the switch if necessary

deutsch

english

français

español

italiano

## 18. General information

Repairs and maintenance work on Labormat devices may only be carried out by qualified personnel. In any case, the power supply to the device must be interrupted. The Dreve Group is only responsible for the safety, reliability and performance of the device when:

- the device is only used for the corresponding designated application.
- operation is only by qualified personnel.

- the device is used in accordance with these operating instructions.
- extensions, new settings, modifications or repairs are only carried out by persons authorised by Dreve.

As we constantly strive to develop our products, we reserve the right to make technical modifications.

## 19. Warranty

The manufacturer warranty is 2 years from date of delivery. The warranty right (liability of defects) is not limited by the warranty and is regulated in the general terms and conditions of the company Dreve. The guarantee includes the assumption of repair and shipping costs in the case of material and manufacturing defects. The warranty claims do not refer to natural wear nor to damages which are the result of the transfer of risk due to faulty or unintended use, excessive loading, unsuitable equipment and any such chemical, electrochemical or electrical influences which are not provided in accordance with the contract. Warranty claims are excluded in the case

of changes or repair and maintenance work which have been incorrectly completed by the purchaser or third parties as well as in the case of the breaching of the seals. All technical changes to the machine are prohibited and will result in the loss of the warranty claim. Repairs may only be performed by trained personnel of the company Dreve. For cases of repair, please contact the company Dreve, Max-Planck-Str. 31, 59423 Unna / Germany; Phone: +49 2303 8807-0. After the sending-in of the product and the submission of the sales receipt, the product will be repaired free of charge or replaced by a comparable product.

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