Destinazione d'uso

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Intended use

Cleaning tablet for external cleaning of dental splints.

(Recommended) Indication

Cleaning product for preservation of dental splints.

Contra indication

The usage of these cleaning tablets is only applicable for dental splints; if any allergy to mint oil exists it may come to an allergic reaction.

Patient target group

Users of dental splints

Intended users

Users of dental splints as well as qualified personnel.

Recommendation for use

Put the cleaning tablet into the MaxiFresh™ bowl filled with 150 ml of water with a temperature of 37 °C.

Please note: In cooler water the cleaning tablet might not dissolve completely. Insert the sieve insert into the cleaning bowl, place the splint to be cleaned into the sieve insert. After the dissolution of the tablet, the reaction time should be at least 15 minutes, the maximum should be 60 minutes.

When reaction time has passed take the dental splint out of the MaxiFresh™ bowl and – with the help of the insert – rinse thoroughly with clear water. After that, dry the surface, you can use a standard, clean paper towel.

Batch number / Use-by date
The batch number and the use-by date are marked on the outer packaging. When submitting a complaint about a

Hazards identification

Hazard and safety information can be found on the product label and/or in the corresponding safety data sheet.

Advice

Shouldn’t you be able to perform the application as described, please get in contact with your hearing aid retailer. Always make sure that you store the MaxiFresh™ cleaning tablets in such a way that they are out of reach of children. Do not ingest or swallow the cleaning tablets!

Serious incidents

All serious incidents occurring in connection with these products or the use of these products must be reported to the manufacturer and the responsible authority.

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