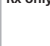
Glossar / Glossary / Glossaire / Glosario / Woordenlijst / Glossario / Glossário

	CE-Kennzeichnung zur Konformität gemäß den Anforderungen an Medizinprodukte / CE marking of conformity according to the requirements for medical devices / Marquage CE de conformité selon les exigences applicables aux dispositifs médicaux / Mercado CE de conformidad según los requisitos de inscripción de productos médicos / CE-markering voor conformiteit conform de eisen aan medische hulpmiddelen / Marcatura CE di conformità secondo i requisiti per i dispositivi medici / Marcação CE de conformidade de acordo com os requisitos para dispositivos médicos
	Temperaturbegrenzung / Temperature limitation / Limite de température / Limitación de temperatura / Temperatur gelimiteerd / Limitazione di temperatura / Limitação de temperatura
	Verwendbar bis / Use-by Date / Date d'expiration / Fecha de caducidad / Te gebruiken tot / Data di scadenza / Data de validade
	Hersteller / Manufacturer / Fabricant / Fabricante / Fabrikant / Fabricante / Fabricante
	Artikelnummer / Catalogue number / Numéro de catalogue / Número de artículo / Artikelnummer / Numero d'articolo / Número do artigo
	Chargennummer / Batch code / Numéro de lot / Número de lote / Batchcode / Numero di lotto / Código de lote
	Medizinprodukt / Medical device / Dispositif médical / Producto sanitario / Medisch hulpmiddel / Dispositivo medico / Dispositivo médico
	Gebrauchsanweisung beachten / Consult instructions for use / Consultez le mode d'emploi / Observar las indicaciones de uso / Gebruiksaanwijzing raadplegen / Osservare le istruzioni per l'uso / Ver por favor as instruções de uso
	Nicht wiederverwenden / Do not re-use / Ne pas réutiliser / No reutilizar / Niet hergebruiken / Non riutilizzare / Não reutilizar
Rx only	Verwendung nur durch dentales Fachpersonal / Use only by dental professionals / Utilisation uniquement par du personnel dentaire spécialisé / Solo podrá ser utilizado por personal dental especializado / Gebruik alleen door gekwalificeerd tandheelkundige personeel / Utilizzo solo da parte di personale dentale qualificato / Use apenas por pessoal dentário qualificado
	Vor Sonneneinstrahlung schützen / Protect from sunlight / Protéger contre les rayons de soleil / Proteger de la radiación solar / Beschermen tegen zonlicht / Proteggere dai raggi solari / Proteger da luz solar

37440/0524 - 20240201–35961

Stand der Information / Date of information / État de l'information / Versión de la información / Laatste wijziging / Stato delle informazioni / Data da informação: 2024-02-01

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Indicatie

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Ethylacetat/Methylmethacrylat, Xylol

Intended use

Adhesive for dental prostheses

Indication for use

Adhesive primer for adhesion between denture relining material and PMMA acrylic resins.

Contraindication

Normally, soft-relining adhesive do not cause allergic reactions; however, an allergic reaction cannot be ruled out in sensitive indi-viduals. If in doubt, we recommend performing an allergy test before use. Do not use the material in case of known allergies to any of the ingredients or in case of contact allergies.

Potential Allergenic Substances

no potential allergenic substances

Target Patient Group

Patients being treated in the context of a dental procedure.

Intended users

For dental professional use only.

Processing

The adhesive is applied uniformly thinly and comprehen-sively to all smoothened, clean areas of the prosthesis where MegaBase® is to adhere, twice at an interval of about 30 sec. Let dry for at least 1.30 minutes. Only continue working once the adhesive is completely dry.

Disinfecting

See the MegaBase® instructions.

Batch number / use-by date

The batch number and the use-by date are located on both the outer packaging and the product. When submitting a complaint about a product, please always quote the batch number. Do not use the product after expiration of the use-by date.

Cautions

• Do not use MegaBase® adhesive with condensation-curing silicones, polyether or polysulfide materials.

• The product is exclusively intended for the above-mentioned range of use.

• For dental professional use only.

• Should the contents of the bottle be spilled, it is recommended that the room be ventilated and the spilled liquid absorbed with a universal binding agent.

• Store in a dry place at room temperature; do not expose to sunlight and avoid exposure to extreme temperature fluctuations. Close the bottle immediately after use.

Keep away from children!

Disposal

Disposal of contents / container in accordance with local / regional / national / international regulations and safety data sheet.

Hazard identification

Hazard and safety notes can be found on the product label and the corresponding safety data sheet.

Srious incidents

All serious incidents occurring in relation to the product shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is settled.

Composition

Ethyl acetate/methyl methacrylate, xylene

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This product was developed for dental use and has to be applied according to the instructions. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. Furthermore the user is bound to check the product according to suitability and possible use in self dependence before using it subject to its purpose, especially in case the purpose is not mentioned in the instructions.

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