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**English**  
**Working Instructions** (EN)

**Modralit® 3K**

**Application:**  
 Filled, polyurethane based preparation material **Modralit® 3K** is excellently suitable for the manufacture of precision models for the whole precious metal and non precious metal technique, especially for the dimension stable reproduction of step- and inlay or onlay preparations

**Characteristics**  
**Modralit® 3K** is pourable, easily flowable, draws exactly, is stable in dimension, edge - and abrasion resistant, easily grindable and sawable.

**Information on processing**  
 Before processing, the glass bottles with basis material and the hardener are shaken. The screw caps are colour marked and therefore cannot be mixed-up. After use the glass bottles have to be closed thoroughly.

<b>REF</b> D3926	<b>Modralit® 3K, Set</b>
<b>REF</b> D39261	<b>Modralit® 3K, 100 ml Basismaterial</b>
<b>REF</b> D39262	<b>Modralit® 3K, 100 ml Härter</b>
<b>REF</b> D39263	<b>Modralit® 3K, 400 g Füllstoff</b>
<b>REF</b> D4305	<b>Separator, 50 ml</b>

<b>Mixing ratios</b>	
Basis liquid	10 ml
Filler	1 Dosierlöffel (ca. 14 g)
Hardener liquid	5 ml

In the case of need the applied quantity can be varied according to the mixing ratio.  
 2. After a mixing time of about 30 sec. let flow **Modralit® 3K** as a thin stream in the cleaned impression. The flowing phase lasts 2 min. at 20°C (68°F).  
 3. In order to avoid air bubbles, after pouring out, **Modralit® 3K** can be hardened in a dry pressure pot at 2–4 bar. After one hour **Modralit® 3K** is hardened so far that the tooth arch can easily be trimmed, drilled and grinded.

Stand der Information: 04.2017 / CL

**English**

4. **Modralit® 3K** adapts all known preparation- and distance lacquers and is resistant towards means of insulation.

**Important indications**  
 • **Modralit® 3K** is available in the primary colour ivory.  
 • Do not expose **Modralit® 3K** to unnecessary warmth. Use stream jet for cleaning (not longer than 15 sec.). A thoroughly controlled processing offers you a preparation of first-class quality.  
 • If using polyether impression resp. duplicating mass a releasing agent (Seperator D4305 or Duosep D4306) has to be used.  
 • If using hydrocolloid-materials the area which has to be filled with **Modralit® 3K** has to be carefully dried with pressure air.  
 • Can cause allergic reactions at sensitive persons in reference to some materials.  
 • A disinfecting is possible with all current materials.  
 • The product is exclusively designed for the field of application described above and only to be used by qualified and instructed persons.  
 • The indications for this product are based on intensive, application-technique experience and research. Due to reasons of development we subject to alterations for the product.

**Modralit® 3K Hardener**  
**Hazard statements**  
**H332** Harmful if inhaled.  
**H315** Causes skin irritation.  
**H319** Causes serious eye irritation.  
**H334** May cause allergy or asthma symptoms or breathing difficulties if inhaled.  
**H317** May cause an allergic skin reaction.  
**H351** Suspected of causing cancer.  
**H335** May cause respiratory irritation.  
**H373** May cause damage to organs through prolonged or repeated exposure.  
**Precautionary statements**  
**P260** Do not breathe dust/fume/gas/mist/vapours/spray.  
**P284** In case of inadequate ventilation wear respiratory protection.  
**P280** Wear protective gloves/protective clothing/ eye protection/face protection.  
**P305+P351+P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.  
**P405** Store locked up.  
**P501** Dispose of contents/container in accordance with-local/regional/national/international regulations.

<b>REF</b> D3926	<b>Modralit® 3K</b>
<b>REF</b> D39261	<b>Modralit® 3K, 100 ml base</b>
<b>REF</b> D39262	<b>Modralit® 3K, 100 ml hardener</b>
<b>REF</b> D39263	<b>Modralit® 3K, 400 g filling material</b>
<b>REF</b> D4305	<b>Separator, 50 ml</b>

This product was developed for dental use and has to be used according to the instructions. In case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. Furthermore the user is bound to check the product according to suitability and possible use in self dependence before using it subject to its purpose, especially in case the purpose is not mentioned in the instructions.

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