# Separator

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# Separator

## Directions for use

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### Important advices:

- With the Separator a separation of polyether impressions to addition-vulcanising silicones is not possible.
- Do not spray against a flame or onto glowing objects.
- Sensitive persons can show allergic reactions to one of the product components.
- The product is exclusively designed for the a. m. field of application and is only to be used by qualified and instructed persons. The general information of this product is based on detailed application practice and intensive research work. Due to the further development we reserve the right of modification.

**Available in:** 50 ml bottle with brush insert and spray head.  
**REF** D4305

**Contains:** Hydrocarbons, C7-C9, n-Alkane, Isoalkane, Cyclene

### Hazard Statements

**H225** Highly flammable liquid and vapour. **H336** May cause drowsiness or dizziness. **H304** May be fatal if swallowed and enters airways. **H411** Toxic to aquatic life with long lasting effects.

### Precautionary statements

**P210** Keep away from heat, hot surfaces, sparks, open flames and other ignition. Use explosion-proof [electrical/ventilating/lighting] equipment. **P301+P310** IF SWALLOWED: Immediately call a POISON CENTER/ doctor. **P303+P361+P353** IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water [or shower]. **P405** Store locked up. **P501** Dispose of contents/container in accordance with local/regional/national/international regulations.

Dreve Dentamid GmbH is not liable for any damages caused by improper application of the material

DANGER

2022-12-19 / Rev. 00  
Dreve Dentamid GmbH  
Max-Planck-Str. 31 • 59423 Unna / Germany  
Tel.: +49 2303 8807-0 • Fax: +49 2303 8807-55  
www.dentamid.dreve.de