Unisol W

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Produktbeschreibung

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Directions for use

Product Description

Additive for the improved wax-solving for the use in boiling-out units (non-foaming, non-emulsifying).

Dosage

According to the water filling quantity of the boiling-out unit Unisol W is dosed as 1 % concentration (e.g. 500 ml to 50 ltr. water).

Field of Application

Add Unisol W to the cold water bath and heat up as usual. Activate the spraying function of the boiling-out unit not until the working temperature is achieved!

Contents: Non-ionic surfactants, preservatives, dyestuff

Available size

REF D4392 2 x 1 ltr. bottle

REF D4393 5 ltr. canister

Unisol W is admitted by the Umweltbundesamt "Federal Agency for Environment" according to the Wasch- und Reinigungsmittelgesetz „guideline for washing and cleaning agents" UBA: 5062 0004.

Possible dangers

Before use observe the Material Safety Data Sheet of Unisol W.

Hazard statements

H319 Causes serious eye irritation.

Precautionary statements


P280 Wear eye protection / face protection. P264 Wash thoroughly after handling.

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P337+P313 If eye irritation persists: Get medical advice/attention.

Lot number / Expiry date: The lot number and the best before date are indicated on each Unisol W packaging. In case of claims please always indicate the lot number of the product. Do not use the product after expiry date.

Dreve Dentamid GmbH is not liable for any damages caused by improper application of the material.

2018-07-19 / Rev. 1
Dreve Dentamid GmbH • Max-Planck-Str. 31, 59423 Unna / Germany
Tel.: ++49 2303 8807-0 • Fax: ++49 2303 8807-55
dentamid@dreve.de • www.dreve.com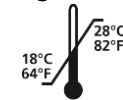

WARNING