

Separator



Technical Data Sheet

Product description:	Wax-based separating agent for not compatible materials, e. g. Modralit® 3K / polyether impressions, A-silicones / C- silicones or for the same kind of materials, e. g. A-silicones / A-silicones.
Manufacturer:	Dreve Dentamid GmbH Max-Planck-Straße 31 59423 Unna / Germany
Chemical characterisation:	Dispersion of waxes in a solvent mixture
Properties:	
Colour:	white
Odour:	typical of the species
Viscosity:	thin-flowing
Density (68 °F):	0,72 g/cm ³
VbF:	AI, highly flammable
Time to drying:	1-2 min
Application:	Use undiluted. Shake before use. Apply the release agent thinly and evenly. Ensure that all areas are evenly wetted with the release agent.

Dreve Dentamid GmbH

Max-Planck-Str. 31

59423 Unna / Germany

Tel.: 02303 / 8807-0

Fax: 02303 / 8807-55

dentamid@dreve.de

www.dreve.com