**Lieferformen / Forms of delivery / Modes de livraison / Formas de entrega / Leveringsvormen / Imballaggio / Formulários de entrega**

**Ø 120 mm**

**klar transparent/clear transparent/clair transparent/claro transparente/helder transparant/chiaro-transparente/transparente claro**

2 mm (1,0 mm hart/1,0 mm weich) (1,0 mm hard/1,0 mm soft) (1,0 mm dur/1,0 mm souple) (1,0 mm hard/1,0 mm zacht) (1,0 mm duro/1,0 mm blando) (1,0 mm rígido/1,0 mm morbido) (1 mm rígida/1 mm mole)	10	  REF D420016
3 mm (1,5 mm hart/1,5 mm weich) (1,5 mm hard/1,5 mm soft) (1,5 mm dur/1,5 mm souple) (1,5 mm hard/1,5 mm zacht) (1,5 mm duro/1,5 mm blando) (1,5 mm rígido/1,5 mm morbido) (1,5 mm rígida/1,5 mm mole)	10	  REF D420017
4 mm (3,0 mm hart/1,0 mm weich) (3,0 mm hard/1,0 mm soft) (3,0 mm dur/1,0 mm souple) (3,0 mm hard/1,0 mm zacht) (3,0 mm duro/1,0 mm blando) (3,0 mm rígido/1,0 mm morbido) (3 mm rígida/1 mm mole)	10	  REF D420018

(Sondergrößen auf Anfrage lieferbar)  
(special measures can be delivered on request)  
(formats spéciaux disponibles sur demande)  
(Tamaños especiales disponibles a petición)  
(speciale afmetingen leverbaar op aanvraag).  
(misure speciali su richiesta)  
(tamanhos especiais disponíveis mediante pedido).

Drewe Design Rev. 150421\_3809

### Finalidad prevista

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## Indications for use

Dental thermoforming blank for the production of func-tional splints

## Material description

Soft / hard thermoforming blank from polyethylenetereph-thalate and ethylvinylacetate for the production of comfort-able and functional splints. The high comfort of wearing leads to an increased safety and success of the therapy.

## Field of application

Suited for interim splints in case of myoarthropathy (muscular and mandibular discomfort) and accompanying dysfunctions resulting from it, like pressing, bruxism, etc. The splint provides for a very good bite lock and does not cause any tension. Furthermore the production is very easy and there is no need for anchorages, the splint fits very well and secure due to the snap-in effect.

The 3 mm blank is also used as functional splint in the DocSnoreNix® system.

## Processing

All Kombiplast materials are delivered with an insulating foil. This extremely thin foil (0,07 mm) is thermoformed together with the Kombiplast on the model side and guar-antees a decisively improved transparency of the splint. Thus an additional insulation of the model surface is no longer necessary. The heating-up of the Kombiplast foils is done from the hard side (the foil is only suitable for pressure forming units).

Protect against humidity, always close foil bag when packs are in use.

**Lot number / Expiry date:** The lot number and the expiry date are indicated on each Kombiplast soft / hard packaging. In case of claims, please always indicate the lot number of the product. Do not use the product after the expiry date.

The adjustment of the heating and cooling times can either be taken from the survey of thermoforming foils, or can be programmed directly into the unit with the help of the barcode on the Kombiplast packaging.

If a Drufoformat scan or a Drufosmart scan is used, please take note of the special information included in this pack-age.

The blank can be built up with autopolymerising material. Avoid the contact between the soft blank and PMMA and other materials that contains alcohol.

## Cleaning

For cleaning, the use of MaxiFresh™ cleaning tablets is recommended.

Please be aware of the MaxiFresh™ instruction for use.

## Possible dangers

Before use observe the Material Safety Data Sheet of Kombiplast soft / hard.

## Storage life

2 years in closed original packaging

This product was developed for dental use and has to be used according to the instructions. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. Furthermore the user is bound to check the product according to suitability and possible use in self dependence before using it subject to its pur-pose, especially in case the purpose is not mentioned in the instructions.

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